

# MOBILE BOVINE PREGNANCY KIT


## Components

No.	Product Name	Spec.
1	FIA scanner	1pc
2	Mini centrifuge	1pc
3	Timer	1pc
4	Centrifuge tube shelf	1pc
5	3mL Vacuum tube (red top)	30pcs
6	20G Disposable venous blood collection needle (yellow and milky-white caps)	30pcs
7	Needle holder	5pcs
8	200μl Pipette tip	96pcs
9	Pipette (20-200μl)	1pc
10	1.5mL Centrifuge tube	30pcs
11	Bovine Pregnancy Rapid Test Kit (Fluorescence)	25T

(\*The components are not confined to fluorescence test kits)




## Operation diagram

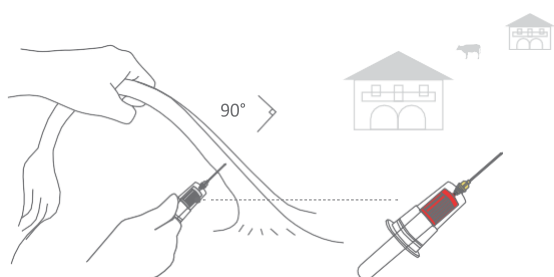


### STEP 1 Blood collection

Use the Blood Sample Collection Kit to take the whole blood. It is recommended that the blood collection site is the center position in the groove about 8cm (the length of vacuum tube) from the cows' tail root.

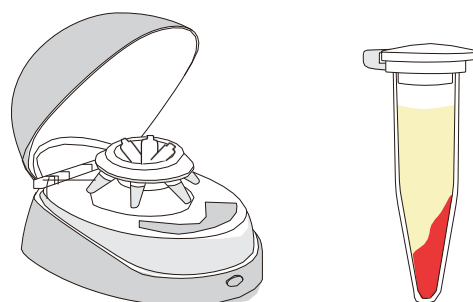


### STEP 2 Blood preparation

Use the Mini Centrifuge to centrifuge for 30-60 seconds to get the serum

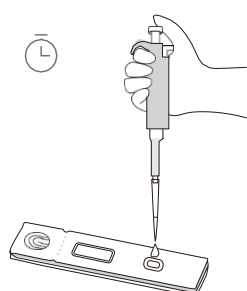

### STEP 3 Detection

Pipete 100μL of serum into the sample well of the test card. Incubate for 15 min at room temperature (18-35°C).

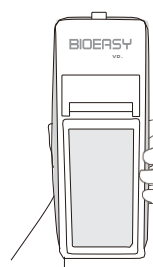

### STEP 4 Result interpretation

Insert the test card into the FIA scanner with the sample well inward, and choose "Quick Test". It will display the test result after a few seconds.