

Sheep/Goat Pregnancy

Rapid Test Kit

Background

Early pregnancy detection in sheep/goat is an important part of the production process in the pasture. Using an accurate, fast, and easy pregnancy diagnosis method is the key to improve breeding efficiency. Aneasy sheep/goat pregnancy rapid test kit can quickly identify pregnant sheep/goat and open sheep/goat on the 35/28 day after post-breeding. Helps to shorten the calving interval and improve calving rates.

***"Sheep: after 35 days of post-breeding
Goat: after 28 days of post-breeding"***

—— Detect early pregnancy in sheep/goat



Advantages



◆ **High sensitivity:** Prepared by latex immunochromatography technology with high sensitivity, higher than that of colloidal gold, and the detection sensitivity is more than 99%.




◆ **Good repeatability:** The marker is stable, strong anti-interference, good repeatability of detection results.




◆ **Easy to use:** No special device is needed and the results can be read visually. Goat: after 28 days of post-breeding; Sheep: after 35 days of post-breeding.

Test Procedure


- 1** Take whole blood and centrifuge to serum or plasma

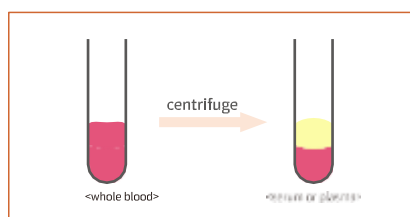

- 2** Add 3 drops (90 μ L) of sample into the sample well

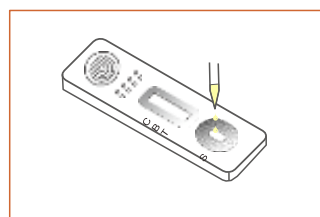


- 3** Incubate at 18~35°C for 20 min

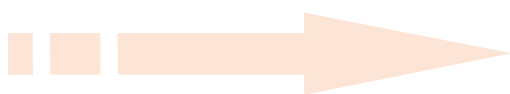

Note: Please interpret the result within 30 mins after the incubation.

- 4** Result interpretation

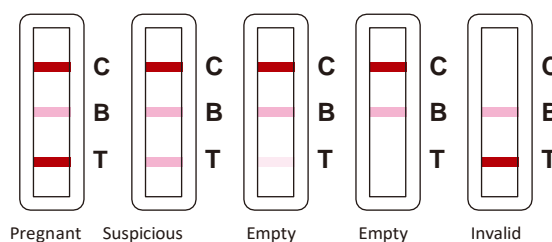

*Suspicious: It is recommended to take the blood again 7 days later and repeat the test.

Product Information

Art No.	Product name	Spec
AE1SGPR001	Sheep/Goat Pregnancy Rapid Test Kit(Latex microspheres)	25T/kit